


# How to collect your sample



**1** Sanitize your hands.

---

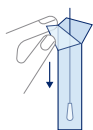

**2** Open the swab package. Avoid touching the cotton tip!

---




**3** Swirl the swab tip three times in both nostrils.

---



**4** Put the swab in the tube, cotton tip down.

---

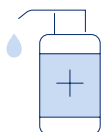

**5** Sanitize your hands again. Done!

# How to activate a COVID-19 test kit

1

Activate your kit online at [color.com/covid/activate](https://color.com/covid/activate).

You need your unique barcode, which you'll find on your collection tube.



**!** **Warning:** Your sample will not be tested if the kit is not activated. If you are under 18, please ask a parent or guardian to help with returning your kit.

2

To activate your kit you will need to create a Color account.

1. Click + Add Dependent.
2. Enter the dependent's information.
3. Enter the dependent's student ID number. If you do not know their student ID, ask your school's front office team.
4. This step ensures that we are connecting your account with the correct program.

For Students <14 years of age, parents or guardians will need to activate an account using the parent/guardian name and email address, with the dependent's Student ID to activate their kit.

For Students >14 years of age, an account can be created using their name and email address, along with their Student ID to activate their kit.

For Staff, an account should be created with their school designated email address.